

ASSEMBLY INSTRUCTIONS

Zido™ Cart




Product	Part#
Zido Carts	ZD2231SB/CG4
	ZD2235SB/CG4
	ZD2240SB/CG4
	ZD2531SB/CG4
	ZD2535SB/CG4
	ZD2540SB/CG4
	ZD2166SB/CG4
Zido Pole Cart	ZP2166SB/CG4



Anthro Corporation® | 10450 SW Manhasset Dr. | Tualatin, OR 97062
Toll-free: 800.325.3841 | Fax: 800.325.0045 | email: sales@anthro.com | anthro.com
Outside the U.S. | Tel: 503.691.2556 | Fax: 503.691.2409

March 2007

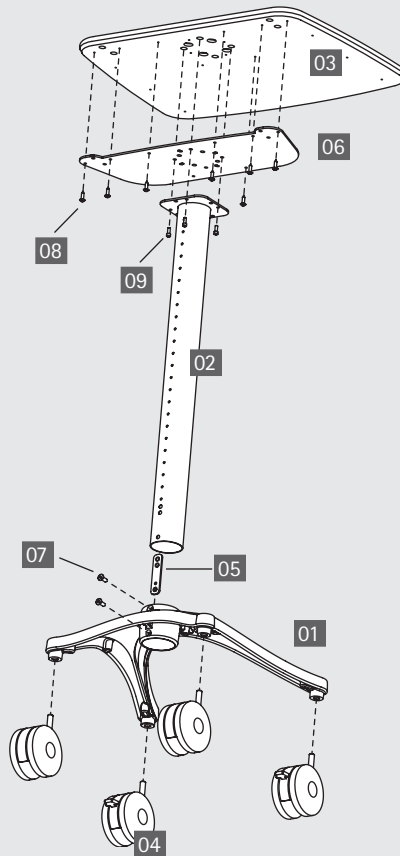

WELCOME

Thank you for purchasing Zido. If you have any questions or if we can help you in any way, please contact us at 800.325.3841.

PARTS LIST

Before beginning assembly of your Zido cart please review the parts list to verify that your shipment is complete.

All Screw quantities listed here are the minimum needed for your Zido assembly. There may be a few extra Screws included which are not counted in the Parts List.

To make the assembly of your Zido even easier, we have included the required tools. The handy Hex Driver Bit can be used in your electric drill in place of the Hex Driver.

Product	Quantity	Part Number
10 Hex Driver 5/32" (not shown)	1	375-5000-00
11 Hex Driver Bit 5/32" (not shown)	1	375-5003-00

Zido Cart		Quantity	Part Number
01	Base		
	Small, 22" wide	1	250-3064-00
	Large, 25" wide	1	250-3065-00
02	Tube		
	24" long (for 31" high cart)	1	500-1031-00
	28" long (for 35" high cart)	1	500-1030-00
	33" long (for 40" high cart)	1	500-1032-00
03	Top		
	22" wide	1	101-6025-00
	25" wide	1	101-6026-00
04	Casters		
	4" Locking	2	150-5011-03
	4" Non-locking	2	150-5010-03
05	Tapped Strip	1	225-2621-00
06	Cap Shelf Bracket	1	225-2620-00
07	3/4" Socket Head Cap Screw	2	325-5461-00
08	Wood Screws	8	325-5106-00
09	1/2" Socket Head Cap Screw	4	325-5455-00
Zido Pole Unit		Quantity	Part Number
01	Base	1	250-3065-00
02	Tube	1	125-4324-00
03	Casters		
	4" Locking	2	150-5011-03
	4" Non-locking	2	150-5010-03
04	Nut Bar	1	225-2621-00
06	3/4" Socket Head Cap Screw	2	325-5461-00
09	End Cap (not shown)	1	175-5235-00

ASSEMBLY SEQUENCE


Plan out the assembly sequence to make your installation easier.

1. Pre-drill the top for accessories (Mouse Holder, Side Bin, Gel Cup/Probe Holder, Extension Tube).
2. Assemble the cart.
3. Base Storage Shelf.
4. Drawer, Keyboard Drawer, Handle.
5. Additional Shelves, Equipment Shelves, Metal Bin, IV bracket.
6. Cable Bin, Glove Box Holder, Tank Holder.
7. Mouse Holder, Side Bin, Gel Cup & Probe Holder, Extension Tube.
8. Pole Cart Handle, Flat Panel Mounts.

STEP 1

Connect the Base with the Tube


- Insert the tube into the base so that holes on the tube line up with the holes on the base and the tube goes all the way to the bottom of the base.
- Slide the nut bar into the bottom of the base assembly, smooth side against the inside of the tube, and line up the holes on the base, tube, and bar. Secure with two 3/4" Socket Head Cap Screws.

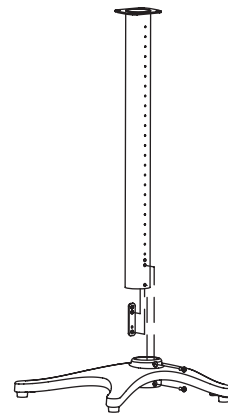

STEP 2

Install the Casters


- Add the locking casters (the ones with the brakes) to the front of the base assembly.
- Add the non-locking casters to the back of the base assembly.
- If you have a Zido™ Pole Cart, insert the End Cap in the top of the Pole. Congratulations! Your assembly is complete!

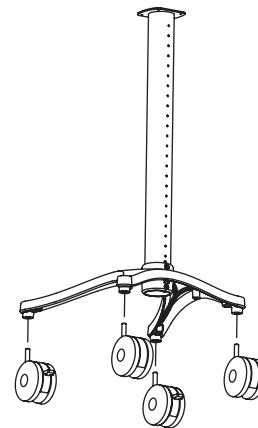

STEP 3

Assemble the Top

- Place the top face down on a non-marring surface. Set the bracket into place so that the off-sets on the bracket nest into the holes in the top.
- Some of the holes on the bracket are threaded and some are not. Using the non-threaded holes, attach the bracket to the top using eight wood screws.
- If you have purchased a Zido Handle for this configuration, install it now. If any of your accessories require that you drill the shelf, do that now.

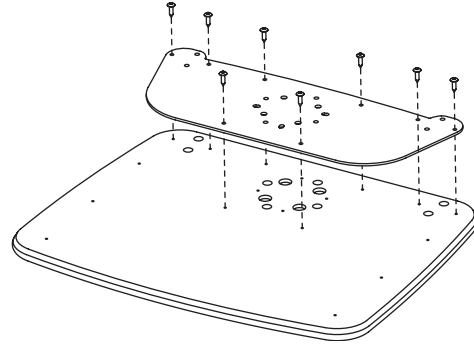


STEP 4

Install the Top Assembly

- Set the top assembly onto the top of the base assembly so that the top extends over the front of the base.
- Secure with four 1/2" socket head cap screws. Be sure to drive the screws by hand using the ball driver provided.

Congratulations! Your assembly is complete!

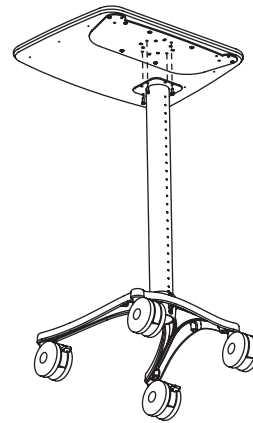



Anthro Corporation® | 10450 SW Manhasset Dr. | Tualatin, OR 97062
Toll-free: 800.325.3841 | Fax: 800.325.0045 | email: sales@anthro.com | anthro.com
Outside the U.S. | Tel: 503.691.2556 | Fax: 503.691.2409



Warranty:
The Zido Carts have a Lifetime Warranty.

Notices:
Technology Furniture and Zido are trademarks of Anthro Corporation. Anthro reserves the right to modify the design and specifications without prior notice.